

Upholstery Cleaning and Maintenance Instructions

- Wipe clean regularly with a micro-fibre cloth to remove dust particles and light soiling.
- Remove spillages promptly with an absorbent dry cloth.
- For general soiling, use a micro-fibre cloth (taking care to avoid abrasive damage to surface) and tepid soapy water (non-alkaline).
- Rinse clean with water and dry with an absorbent cloth.
- Bodily fluids should be removed promptly (within 15 minutes) with cold water and then cleaned as above.
- Extreme soiling/ infection control purposes use a 5% sodium hypochlorite (bleach) solution, Haz Tabs or Chlorclean.
- Steam cleaning is an effective sanitizer (as are Haz Tabs and Chlorclean) but is not a good cleaner. A wipe with a cloth, preferably micro-fibre, and water is more effective for actual cleaning. Steam and heat from steam cleaners actually seal in oily residues from bodily fluids, etc. If steam cleaning is essential, use infrequently for emergency infection control purposes only and in accordance with the machine manufacturer's recommendations. As we are unable to comment on the variety of machines in the marketplace or on the operator's handling, care should be taken for the sake of both the upholstery fabric and the wooden frame and we are unable to guarantee these.
- Do not machine wash or dry clean.
- Do not use furniture polish, stain remover or solvents.
- Remove indigo dye from denim and canvas garments immediately as they will produce permanent staining if not dealt with promptly.
- Do not use a felt tip or ball-point pen to mark.

Furniture Cleaning and Maintenance



Always clean the bed before use, in particular when assigning new patients to reduce the risk of any possible cross infection.



Before beginning any cleaning or disinfecting procedure, make sure you are wearing suitably protective clothing and gloves to prevent against skin irritation or other adverse skin reactions.



Always dispose of cleaning agents in compliance with any instructions given with the cleaning agent and facility policies.



Never use abrasive cleaning agents or pads on the bed, doing so will damage the finish of the Bed.



If disinfection procedure involves large scale spraying of flammable disinfectants, special precautions should be taken to prevent against the risk of fire or explosion.



Always dispose of all cleaning agents & waste in accordance with company procedures.